



Berlinger – feel safe

Vreni Müller | Brasília, 14.08.2024





30 years in Doping
Control Business



Sales to over
175 countries



Family- owned



Swiss Quality
Standard



1 Mio. containers
per year



Sustainability
is at the heart of our design process

BEREG-Kit



- Urine Sample Container made of glass
- Most widely used tamper-evident container in anti-doping

BEREG-Kit Plastic



- Urine Sample Container made of plastic
- Available since 2023

BEREG-Kit small



- Blood Sample Container made of plastic
- Available as single container or in a set of 2 or 3 containers

Urine Collection Vessel



- For collection and temporary storage of a urine sample and decanting into the Urine Collection Kit

Partial Sample Bavaria



- For temporary sealing of a partial sample within the kit
- Available for BEREK-Kit and BEREK-Kit Plastic

Partial Sample Bag



- For temporary sealing of a partial sample directly in the Urine Collection Vessel

Handling of Doping Control Equipment

Autor. Brasil. Controle de Dopagem



Berlinger Urine Collection Vessel

General Information

- Stackable
- Sturdy material
- Physical Indication
- Moulded scale
- Screw cap
- Watertight
- Contamination protected
- Lateral spout for guidance



BEREG-Kit Plastic

General Information

- Light weight
- Robust
- Medical grade PET
- Scale on the back
- Consistent handling



BEREG-Kit small

General Information

- Small and light
- Robust
- Consistent handling




Check

- Check the expiry date

Comply

- Comply with temperature limits

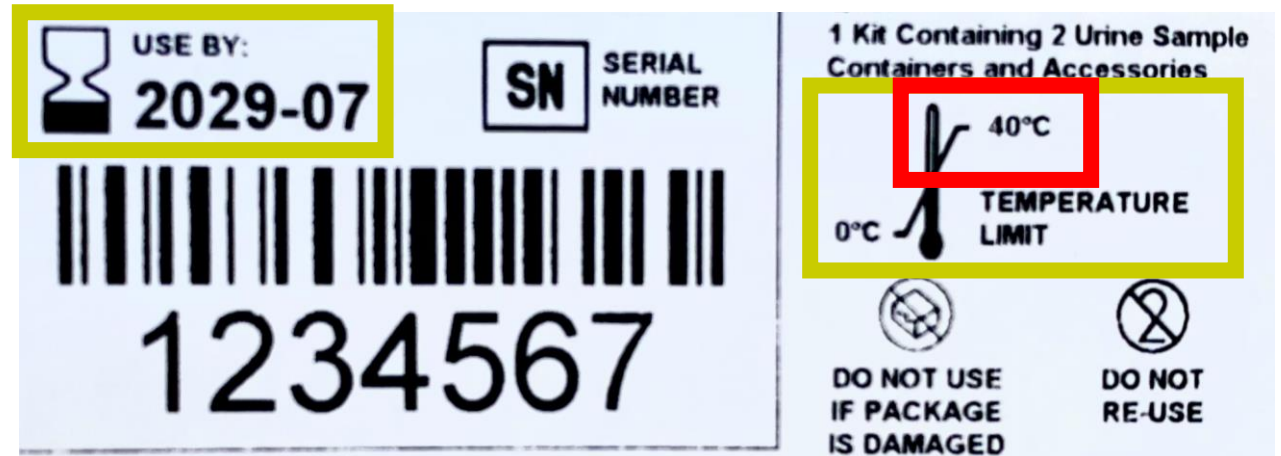

Berlinger Urine Collection Vessel

Handling Instruction

Check

- Check if the packaging shows any signs of manipulation

Open

- Only open the lower part which is marked with "1"

Take out

- Take out the vessel

Don't touch

- Don't touch the inside of the vessel!



Collect

- Collect the urine sample according to guidelines

Place

- Place the vessel on a firm surface

Open

- Open the 2nd chamber of the plastic bag containing the cap

Don't touch

- Don't touch the inside of the cap

Screw

- Screw the cap onto the vessel

Don't touch

- Don't touch the foil that covers the spout

Berlinger Urine Collection Vessel

Handling Instruction



Partial Sample Bag

Currently not used by ABCD – Quick Overview Handling Instruction

Place

- Place the Urine Collection Vessel in the bag

Close it

- Close it by removing the backing paper from the adhesive

Note down

- Note down the number



Open

- Open the kit which you will also use to finally seal the sample

Take out

- Take out only one sample container and remove the cap

Leave

- Leave the red ring on the container neck

Fill

- Fill the collected urine into the container

Put on

- Put on the black plug



Place

- Place the cap back onto the container and the container back into the box

Make sure

- Make sure the kit is in an upright position when putting it into the security plastic bag

Close it

- Close it by removing the backing paper from the adhesive

Note down

- Note down the number



BEREG-Kit Plastic

Handling Instruction

Check

- Check if the packaging shows any signs of manipulation

Pull off

- Pull off the tear-off strip

Check

- Check if the content is complete and all number are identical

Remove

- Remove the shrink sleeve wraps



BEREG-Kit Plastic

Handling Instruction

**Make
sure**

- Make sure that all elements of the caps are in place: White plug + Spring clip + Black locking ring

Remove

- Remove the red rings

Explain

- Explain the fill level indications to the athlete



Berlinger Urine Collection Vessel

Handling Instruction

Remove

- Remove the foil from the spout of the cap

Place

- Place the spout on the edge of the container

Don't

- Don't cover the ventilation hole during decanting



Close

- Close both sample containers

Don't

- Don't use too much force

Inspect

- Inspect the closed container: turn counterclockwise + try to lift + turn upside down



Put


- Put the containers in a watertight bag

Remove

- Remove the backing paper from the adhesif for closing

Place

- Place the containers back into the box and close it

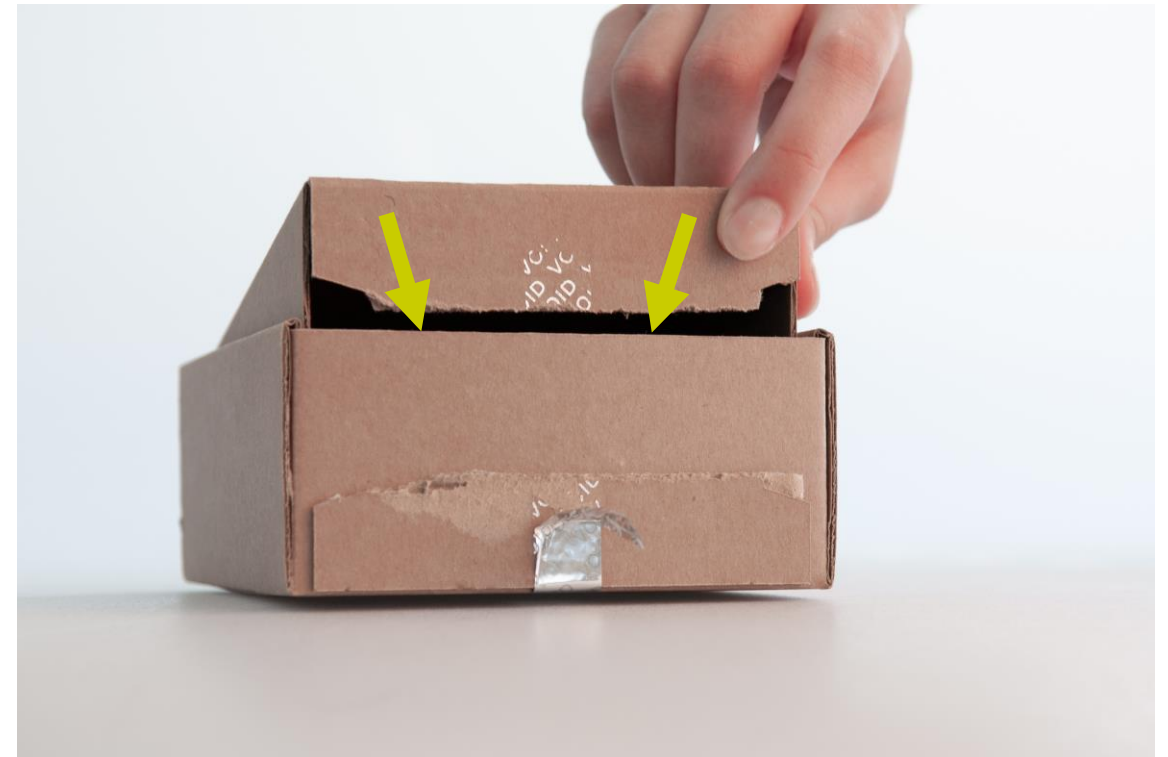

Berlinger Urine Collection Vessel

Handling Instruction

Insert

- Insert the pipette through the spout to get a last drop of urine for specific gravity measurement



Check

- Check if the packaging shows any signs of manipulation

Check

- Check if the content is complete and all number are identical

Remove

- Remove the shrink sleeve wraps

Make sure

- Make sure that all elements of the caps are in place: White plug + Spring clip + Black locking ring

Remove

- Remove the red rings



Insert


- Insert the blood tubes in the containers

Close

- Close the sample containers

Inspect

- Inspect the closed container: turn counterclockwise + try to lift

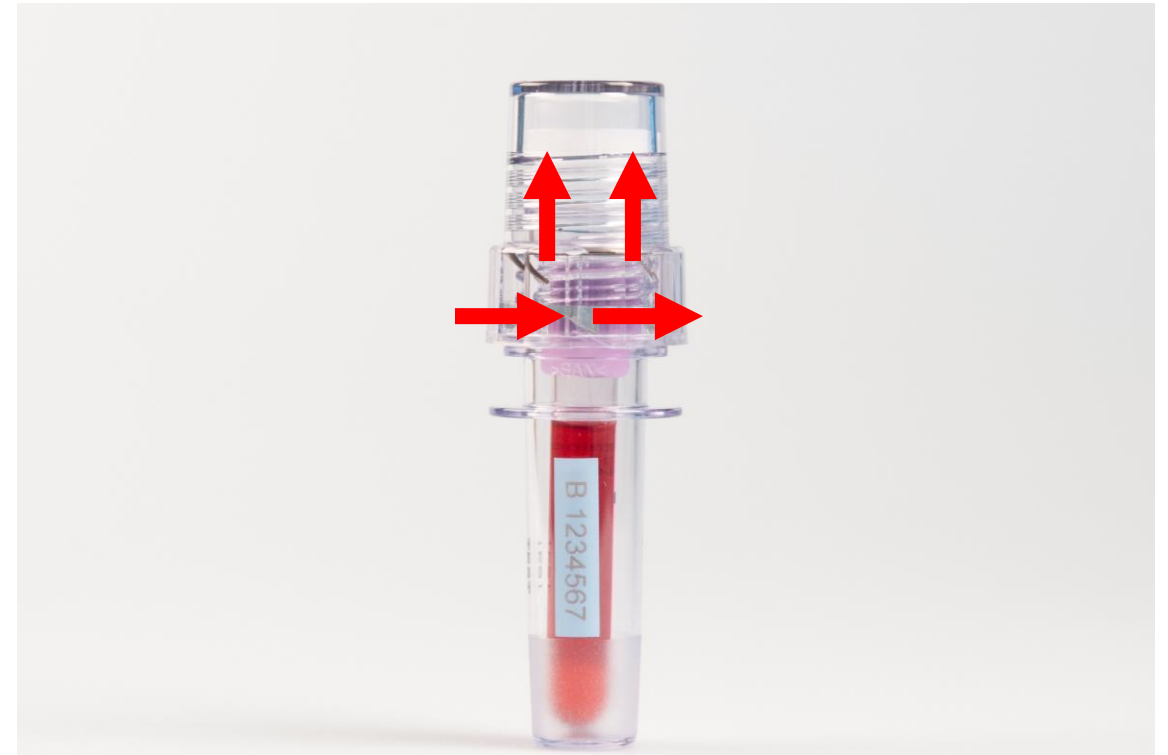

Put

- Put the containers in a watertight bag

Remove

- Remove the backing paper from the adhesif for closing

Place

- Place the containers into suitable cooling box





Follow us

For helpful information and to stay up to date

Berlinger Special AG LinkedIn



Berlinger Special AG Youtube





Thank you

Berlinger – feel safe

